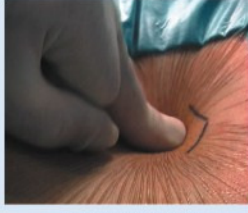
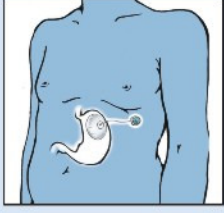
Adjustable Totally Implanted Intra-Gastric Prosthesis

The Adjustable Totally Implanted Intra-gastric Prosthesis (ATIP) - ENDOGAST is a new mini-invasive technique for the treatment of Morbid Obesity.

The aim of the ATIP ENDOGAST is to induce an early satiety interfering with the satiety control process of the corpus-fundus area of the stomach (gastric accommodation and neuro-hormonal mechanisms) and consequently to induce the reduction of meal introduction and weight loss.

The ATIP ENDOGAST is conducted using a surgical and endoscopic procedure. The permanent presence of an air inflated prosthesis inside the gastric corpusfundus and the fixation to the abdominal wall are the two main principles of the technique. The prosthesis is connected to a subcutaneous totally implanted system.

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

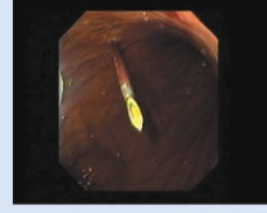
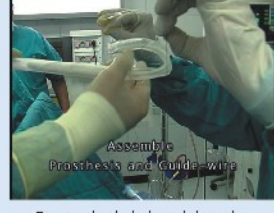
Assemble Prosthesis and Guide-wire

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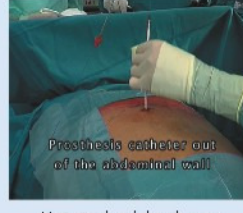
Prosthesis catheter out of the abdominal wall

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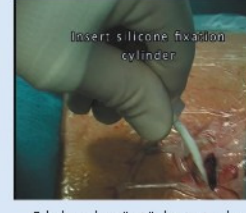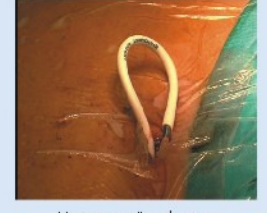Kateter tünel açın

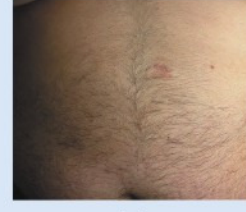
Catheter and titanium port assembled

Kateter ve portu bağlayın

1 Ay

6 Ay

16 Ay

General description of the device: The intra-gastric prosthesis ENDOGAST is medical device implantable in the stomach, set up under endoscopic control and fixed by percutaneous way, for the treatment of severe and morbid obesity. This is a nominal volume device, inflated by air using a catheter implantable Port connected to the prosthesis.

Characteristics: Endo 25D version

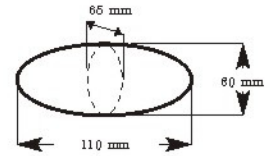
Weight:

Prosthesis = 11 grams

Port with connection ring = 14 grams

Overall dimensions:

Oval prosthesis
inflated at 250ml

Material: Districlass Médical SA ensures and declares that this medical device is « Free Latex », and that the product and/or the process used does not contain any product of animal origin.